

# Brugsanvisning

## Hatting sædpose med tap

Instructions for use  
Hatting semen bag with tip



**Brugsanvisning for Hatting sædpose med tap**  
Instructions for use  
Hatting semen bag with tip



Tag fat om tappen med den ene hånd og tag fat om toppen af posen med den anden. Gennembryd posen med tappen med et fast greb og frigør maximalt 1,5 cm af tappen. **Vigtigt:** Pas på ikke at frigøre for meget af tappen for så sidder den ikke ordentlig fast i posen.

Grab the tip with one hand and grab the top of the bag with the other. Break through the bag with the tap with a firm grip and release a maximum of 1.5 cm of the tap. **Caution:** Be careful not to release too much of the tap because the it is not properly stuck in the bag.



Pres tappen ind i katetret med et fast greb samtidig med at du holder godt ved tappen. Undgå at "skrue" tappen ind i katetret for så risikerer man at tappen går løs i sædposen.

Press the tap into the catheter with a firm grip while holding the tap firmly. Avoid "screwing" the tap into the catheter as there is a risk of the tap coming loose in the semen bag.